

Morform® has been created for easy use. Just follow the simple steps below to experience the benefits.



To gel pouch contents:

1. Place one sealed sachet into your pouch before fitting to your body.



2. Additional sachets may be inserted into drainable pouches after emptying.



3. Morform® is non-toxic and flushes away in the usual manner.



Use only as directed.  
For external use only. Do not eat.

### CliniMed Morform® Motion Management Sachets

Order code	Contents	PIP code
3837	60 sachets	308-6451

[www.clinimedessentials.co.uk](http://www.clinimedessentials.co.uk)

**CliniMed®**  
Essentials ●●●●●



CliniMed Ltd is a CliniMed Group Company.  
CliniMed stoma accessories are distributed in the UK by:  
CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks, HP10 9QY.  
Tel: 01628 850100 Fax: 01628 850331  
E-mail: [enquiries@clinimed.co.uk](mailto:enquiries@clinimed.co.uk) [www.clinimed.co.uk](http://www.clinimed.co.uk)  
N. Ireland distributor: Fannin (NI) Limited. Customer Support 02890 735588  
Email: [info@fanninni.com](mailto:info@fanninni.com)  
Republic of Ireland distributor: Fannin (ROI) Limited. Customer Support 003531 290 7047  
Email: [info@fannin.ie](mailto:info@fannin.ie)  
Morform® and CliniMed® are registered trademarks of CliniMed (Holdings) Ltd.

1088/0509



# Confidence



Motion Management Sachets

# Confidence of a flatter, more secure pouch

## For ileostomates

The main function of the colon is to absorb moisture. If part of the bowel is reduced due to surgery, this can result in a very liquid output, making your pouch bulge and more likely to leak. Morform® absorbs moisture from the pouch contents, converting it to a more manageable gel, which can be drained from your pouch in the normal way. This makes your pouch flatter and less frequent emptying is required.

## For colostomates

An upset tummy can cause a highly liquid output, with all the associated leakage, comfort and appearance problems. Morform® helps combat these problems by gently absorbing moisture in your pouch, making it flatter, quieter and easier to manage, with fewer pouch changes.

*Please note: Morform® is designed to enable ostomates to manage a normally liquid output. If you experience an abnormal change in your output consistency, you should contact your Stoma Care Nurse or GP for advice.*



- reduces pouch bulging
- improves ostomy comfort
- for less frequent emptying
- reduces pouch noise
- reduces pouch odour

## Morform® is ideal for

- social occasions
- shopping trips
- recreational use
- night time use
- everyday use



**Morform® is available on prescription in discreet and convenient soft packs of 60 sachets**

Call our

**Free Careline  
0800 0360100**