

*Freedom to change...
almost anywhere*

Ostigel® skin cleanser gives you the freedom to change your pouch anywhere. Simply apply to a swab or tissue as appropriate and wipe to remove any soiling. Ostigel® dries in seconds allowing you to apply your pouch as usual.

*...No water, no towels,
no fuss*

The antibacterial action of Ostigel® will ensure hygienically clean hands after a pouch change. Remove any soiling with a tissue and apply Ostigel® directly to your hands from the bottle or the sachet, massage into the skin to leave them clean, fresh and moisturised.

Ostigel® contains alcohol and therefore should not be applied directly on to the stoma.



Ostigel.
skin cleanser

is available on prescription in discreet
sachets or 100ml bottles

Call the

Free Careline
0800 0360100



How to use Ostigel®

Simply apply to a swab or tissue and wipe to remove any soiling.



After a pouch change, clean your hands with Ostigel®. No need for water or a towel.



See bottle or sachet for ingredients and safety information.

CliniMed Ostigel® Skin Cleanser

Order Code	Pack size	PIP Code
8801	1 x 100ml bottle	289-8542
8802	30 x 1.5ml sachets	289-8559

www.clinimedessentials.co.uk

CliniMed®
Essentials ●●●●●



CliniMed Ltd is a CliniMed Group Company.

CliniMed stoma accessories are distributed in the UK by:

CliniMed Ltd., Cavell House, Knave's Beech Way, Loudwater, High Wycombe, Bucks, HP10 9QY.

Tel: 01628 850100 Fax: 01628 850331

E-mail: enquiries@clinimed.co.uk www.clinimed.co.uk

N. Ireland distributor: Fannin (NI) Limited. Customer Support 02890 735588

Email: info@fanninni.com

Republic of Ireland distributor: Fannin (ROI) Limited. Customer Support 003531 290 7047

Email: info@fannin.ie

Ostigel® and CliniMed® are registered trademarks of CliniMed (Holdings) Ltd.

1089/0509

Freedom

Ostigel®
skin cleanser